

# Whole Exome Sequencing

## Gene package Autoinflammatory diseases, version 5.1, 25-2-2022



### Technical information

DNA was enriched using the Agilent SureSelectXT Human All Exon V7 capture kit and paired-end sequenced on the Illumina platform (outsourced). Sequencing data are demultiplexed with bcl2fastq2 Conversion Software from Illumina. Illumina DRAGEN Bio-IT Platform is used for read mapping to the hg19 genome and sequence variant detection. The detected sequence variants are annotated and filtered with Alissa Interpret software and classified with Alamut Visual. Copy number variant detection is performed using the BAM multiscale reference method using depth of coverage analysis and dynamical bins in NexusClinical. The detected copy number variants are annotated and filtered with the NexusClinical software and classified using UCSC Genome Browser (NCBI37/hg19). The sensitivity to detect variants using this technology is not 100%; pathogenic variants could be missed. At this moment, there is not enough information about the sensitivity of this technique with respect to the detection of deletions and duplications of more than 5 nucleotides and of somatic mosaic mutations (all types of sequence changes).



Dept. Clinical Genetics

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered $\geq 10x$	% covered $\geq 20x$	% covered $\geq 30x$	% covered $\geq 50x$
ACP5	171640	100	% covered $\geq 10x$	% covered $\geq 10x$	% covered $\geq 10x$
ADA2	607575	100	100	100	100
ADAM17	603639	100	99.70	98.07	91.74
ADAR	146920	98.45	98.45	98.45	98.04
ADGRE2	606100	100	100	98.68	94.77
AP1S3	615781	100	100	97.78	95.91
CARD14	607211	100	99.21	97.60	88.97
CEBPE	600749	100	100	100	99.89
COPA	601924	100	100	100	98.67
DDX58	609631	100	100	100	98.36
DNASE1	125505	100	100	100	97.31
DNASE1L3	602244	100	100	100	98.13
DNASE2	126350	100	100	97.25	88.21
IFIH1	606951	100	100	100	99.35
IKZF1	603023	100	100	100	96.22
IL10	124092	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
IL10RA	146933	100	100	98.77	94.77
IL10RB	123889	100	95.38	93.54	90.04
IL1RN	147679	100	100	100	99.70
IL36RN	605507	100	100	97.80	86.97
LACC1	613409	100	100	100	100
LPIN2	605519	100	100	100	98.57
LSM11	617910	100	93.06	77.89	64.05
MEFV	608107	100	100	99.57	94.81
MVK	251170	100	100	100	96.15
NCKAP1L	141180	100	100	99.60	94.99
NCSTN	605254	100	100	100	98.42
NLRC4	606831	100	100	100	100
NLRP1	606636	100	100	99.99	98.02
NLRP12	609648	100	100	99.26	94.99
NLRP3	606416	100	100	100	99.39
NOD2	605956	100	100	99.81	96.90
OTULIN	615712	91.60	85.62	85.62	82.65
PEPD	613230	100	100	98.93	91.34
PIK3CD	602839	100	100	99.61	98.52
PLCG2	600220	100	100	100	96.79
POMP	613386	100	93.37	85.08	80.85
PRKCD	176977	97.63	97.63	97.63	97.32
PSENN	607632	100	100	98.35	89.94
PSMA3	176843	100	100	100	94.92
PSMB4	602177	100	100	99.57	92.86
PSMB8	177046	100	100	100	100
PSMB9	177045	100	100	99.94	89.70
PSMG2	609702	100	100	100	99.14
PSTPIP1	606347	100	100	100	92.34
RBCK1	610924	100	100	100	91.33
RIPK1	603453	100	100	100	95.68
RNASEH2A	606034	100	100	99.81	92.38
RNASEH2B	610326	100	91.35	91.35	91.35
RNASEH2C	610330	100	100	100	100
RNF31	612487	100	100	99.20	95.41
SAMHD1	606754	100	100	100	100
SH3BP2	602104	91.15	91.15	91.15	88.74
SLC29A3	612373	100	97.60	97.60	97.48

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
STAT2	600556	100	100	100	99.14
STING1	612374	100	100	100	88.83
TNFAIP3	191163	100	100	99.80	98.30
TNFRSF1A	191190	100	100	100	97.54
TREX1	606609	100	100	100	100
TRNT1	612907	100	97.75	89.81	86.79
UBA1	314370	100	99.93	97.88	90.57
USP18	607057	95.21	95.21	95.21	92.59
WDR1	604734	100	99.91	98.30	97.36

- OMIM release used: 23-9-2021

- The statistics above are based on a set of 104 samples

- % Covered 10x , 20x, 30x and 50x describes the percentage of a gene's coding sequence (±10bp flanking introns) that is covered at least 10x, 20x, 30x or 50x