

Whole Exome Sequencing

Gene package Vision disorders, version 10, 30-9-2021



Technical information

DNA was enriched using Agilent SureSelect DNA + SureSelect OneSeq 300kb CNV Backbone + Human All Exon V7 capture and paired-end sequenced on the Illumina platform (outsourced). The aim is to obtain 10 Giga base pairs per exome with a mapped fraction of 0.99. The average coverage of the exome is ~50x. Duplicate and non-unique reads are excluded. Data are demultiplexed with bcl2fastq Conversion Software from Illumina. Reads are mapped to the genome using the BWA-MEM algorithm (reference: <http://bio-bwa.sourceforge.net/>). Sequence variant detection is performed by the Genome Analysis Toolkit HaplotypeCaller (reference: <http://www.broadinstitute.org/gatk/>). The detected sequence variants are filtered and annotated with Alissa Interpret software and classified with Alamut Visual. Copy variant detection is performed using the BAM multiscale reference method using depth of coverage analysis and dynamical bins in NexusClinical. The detected copy number variants are filtered and annotated with the NexusClinical software and classified using UCSC Genome Browser (NCBI37/hg19). It is not excluded that pathogenic variants are being missed using this technology. At this moment, there is not enough information about the sensitivity of this technique with respect to the detection of deletions and duplications of more than 5 nucleotides and of somatic mosaic mutations (all types of sequence changes).



Dept. Clinical Genetics

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered $\geq 10x$	% covered $\geq 20x$	% covered $\geq 30x$	% covered $\geq 50x$
ABCA4	601691	100	100	99.29	94.45
ABCB6	605452	100	100	100	98.61
ABCC6	603234	99.56	98.03	96.73	84.24
ABHD12	613599	99.25	94.29	85.67	80.26
ACBD5	616618	100	99.84	98.09	96.15
ACO2	100850	100	100	100	100
ACTB	102630	100	100	100	99.59
ACTG1	102560	100	100	100	100
ADAM9	602713	100	99.88	98.57	94.41
ADAMTS10	608990	100	100	99.74	94.21
ADAMTS17	607511	95.41	90.13	86.31	82.79
ADAMTS18	607512	100	98.79	97.33	94.59
ADAMTSL4	610113	100	99.10	98.06	92.03
ADGRV1	602851	100	99.96	99.50	96.75
ADIPOR1	607945	100	100	99.72	86.84
AFG3L2	604581	93.67	91.50	91.50	90.44
AGBL1	615496	100	100	100	97.91
AGBL5	615900	100	99.98	98.91	96.53
AGK	610345	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
AHI1	608894	100	100	100	99.14
AHR	600253	100	100	100	99.42
AIPL1	604392	100	100	100	98.19
ALDH1A3	600463	100	99.16	93.37	87.64
ALMS1	606844	99.93	99.38	98.73	95.61
ALX3	606014	91.84	79.58	73.22	73.22
ANTXR1	606410	98.59	97.25	95.93	88.60
AP3B1	603401	97.72	95.97	94.31	85.98
AP3D1	607246	97.71	97.71	96.35	90.76
ARHGEF18	616432	100	98.33	94.49	87.22
ARL13B	608922	100	99.19	97.91	85.75
ARL2	601175	100	100	88.50	82.36
ARL2BP	615407	100	100	100	100
ARL3	604695	100	96.55	86.41	85.89
ARL6	608845	100	100	100	95.53
ARMC9	617612	100	99.97	98.53	90.51
ARR3	301770	100	100	100	91.54
ARSG	610008	100	100	99.78	96.18
ASB10	615054	100	100	100	99.67
ASPH	600582	100	100	100	97.30
ASRGL1	609212	100	100	96.55	86.73
ATF6	605537	100	100	98.00	91.50
ATOH7	609875	100	100	93.70	82.14
ATP1A3	182350	100	100	99.43	95.12
B3GLCT	610308	100	94.98	94.98	94.31
B9D1	614144	100	100	100	99.21
B9D2	611951	100	100	100	100
BBIP1	613605	100	100	100	100
BBS1	209901	100	100	100	99.32
BBS10	610148	100	100	99.55	98.14
BBS12	610683	100	100	100	100
BBS2	606151	100	100	100	100
BBS4	600374	100	100	100	95.74
BBS5	603650	100	100	100	97.62
BBS7	607590	100	100	99.45	96.44
BBS9	607968	99.97	99.59	98.64	93.69
BCOR	300485	100	100	99.79	96.98
BEST1	607854	100	100	100	99.01
BFSP1	603307	100	100	99.65	91.76
BFSP2	603212	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
BLOC1S3	609762	100	100	85.38	47.68
BLOC1S5	607289	100	100	88.80	74.74
BLOC1S6	604310	100	99.45	92.16	78.29
BMP4	112262	100	100	98.65	94.33
BMP7	112267	100	100	100	97.80
BSG	109480	100	97.41	90.87	85.74
C12orf57	615140	100	100	100	100
C19orf12	614297	100	100	100	100
C1QTNF5	608752	100	88.43	73.47	68.73
C3	120700	100	100	99.61	96.57
CA4	114760	100	100	100	97.22
CABP4	608965	100	100	99.37	92.17
CACNA1F	300110	100	99.79	97.43	84.65
CACNA2D4	608171	100	100	99.95	95.91
CAPN5	602537	100	100	100	95.97
CBS	613381	100	100	100	100
CC2D2A	612013	100	99.87	98.46	91.02
CCT2	605139	100	100	100	96.21
CDH2	114020	98.49	97.37	96.35	91.24
CDH23	605516	100	100	100	87.81
CDH3	114021	100	98.57	95.19	92.95
CDHR1	609502	97.43	97.43	97.43	95.41
CEP104	616690	100	100	99.86	97.26
CEP120	613446	100	100	99.84	96.58
CEP164	614848	100	99.73	97.31	87.95
CEP250	609689	100	99.96	99.06	91.44
CEP290	610142	92.60	87.26	83.53	75.20
CEP41	610523	100	100	100	97.30
CEP78	617110	100	100	99.38	93.56
CEP83	615847	100	90.51	82.41	72.45
CERKL	608381	100	100	100	97.88
CFAP410	603191	100	100	98.79	83.09
CFAP418	614477	No coverage data			
CFB	138470	100	100	100	99.51
CFH	134370	99.81	98.52	95.51	92.17
CFI	217030	86.45	83.17	81.24	78.74
CHD7	608892	100	100	99.90	98.10
CHM	300390	91.63	91.63	91.63	91.63
CHMP4B	610897	100	100	100	83.87
CHRD1	300350	100	100	100	91.81

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
CHST6	605294	100	100	100	100
CIB2	605564	100	100	100	97.58
CISD2	611507	87.96	76.99	76.99	76.99
CLCC1	617539	100	99.95	99.08	94.28
CLDN19	610036	100	100	100	100
CLN3	607042	100	97.96	91.76	88.85
CLN5	608102	93.14	93.14	87.18	68.93
CLN6	606725	100	100	99.35	92.17
CLN8	607837	100	100	100	100
CLRN1	606397	100	100	100	100
CLUAP1	616787	100	100	100	85.13
CNGA1	123825	100	100	100	96.91
CNGA3	600053	100	100	100	98.99
CNGB1	600724	99.95	95.83	91.24	74.80
CNGB3	605080	100	100	99.53	95.50
CNNM4	607805	100	98.76	97.89	96.06
COL11A1	120280	100	99.17	97.37	87.82
COL11A2	120290	100	100	98.35	87.06
COL18A1	120328	100	99.83	98.32	86.94
COL25A1	610004	100	100	99.63	92.17
COL2A1	120140	100	100	99.72	94.32
COL8A2	120252	100	99.37	96.72	67.68
COL9A1	120210	100	100	97.67	84.62
COL9A2	120260	100	96.12	88.64	64.54
COL9A3	120270	100	99.22	95.30	79.74
CPAMD8	608841	99.89	97.87	94.30	83.81
CPLANE1	614571	100	99.26	97.64	95.07
CPSF1	606027	99.23	98.62	97.89	92.95
CRB1	604210	100	100	100	100
CRX	602225	100	100	100	95.72
CRYAA	123580	100	100	100	100
CRYAB	123590	100	100	100	100
CRYBA1	123610	100	100	100	100
CRYBA2	600836	100	100	98.73	80.63
CRYBA4	123631	100	100	100	91.64
CRYBB1	600929	100	100	100	99.59
CRYBB2	123620	100	100	100	100
CRYBB3	123630	100	100	100	100
CRYGB	123670	100	100	96.24	89.40
CRYGC	123680	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
CRYGD	123690	100	100	100	99.14
CRYGS	123730	100	100	100	100
CSPP1	611654	100	97.28	95.13	90.44
CTDP1	604927	99.95	94.29	88.29	81.86
CTNNA1	116805	100	100	99.02	94.88
CTNND1	601045	100	100	100	97.99
CTSD	116840	99.58	93.79	93.79	91.28
CTSH	116820	100	100	100	97.65
CWC27	617170	96.26	89.68	88.92	77.39
CYP1B1	601771	100	100	100	100
CYP4V2	608614	100	100	100	97.13
DCN	125255	100	100	100	99.59
DHDDS	608172	100	100	100	94.35
DHX38	605584	100	100	98.98	91.32
DKC1	300126	100	99.10	97.39	86.54
DNAJC30	618202	100	100	100	100
DNM1L	603850	100	100	99.23	91.86
DNMBP	611282	100	100	100	99.64
DPYD	612779	100	100	100	100
DRAM2	613360	100	100	100	99.89
DTHD1	616979	100	98.99	96.83	95.13
DTNBP1	607145	100	100	100	95.91
DYNC2I2	613363	100	100	97.93	83.59
EFEMP1	601548	100	100	100	99.82
ELOVL1	611813	100	100	100	99.08
ELOVL4	605512	100	100	100	95.67
ELP4	606985	95.99	93.14	90.84	86.07
EMC1	616846	99.16	99.16	99.16	97.58
EPG5	615068	100	99.91	99.15	95.77
EPHA2	176946	100	100	98.91	94.46
EXOSC2	602238	100	100	100	99.06
EYA1	601653	97.62	97.62	97.62	94.11
EYS	612424	100	100	99.46	93.42
FA2H	611026	100	95.14	87.98	73.33
FAM161A	613596	100	99.67	97.55	91.13
FBN1	134797	100	100	100	99.50
FLVCR1	609144	100	100	98.87	92.92
FOXC1	601090	97.11	90.56	84.87	73.23
FOXE3	601094	83.06	73.13	61.67	40.89
FOXL2	605597	100	95.30	90.20	76.57

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
FRAS1	607830	100	100	99.39	94.71
FREM1	608944	100	100	99.99	98.70
FREM2	608945	100	100	99.98	99.09
FRMD7	300628	100	100	100	100
FTL	134790	100	100	97.19	88.02
FYCO1	607182	100	100	100	98.84
FZD4	604579	100	100	100	99.58
GALK1	604313	100	100	100	98.47
GALM	137030	100	100	100	100
GALT	606999	100	100	100	98.14
GCNT2	600429	100	100	100	100
GDF3	606522	100	100	100	99.20
GDF6	601147	100	100	99.07	74.80
GDPD1	616317	100	100	100	99.12
GFER	600924	100	100	99.78	88.15
GJA1	121014	100	100	100	100
GJA3	121015	100	100	100	84.04
GJA8	600897	100	100	100	97.50
GLI2	165230	100	98.72	96.47	88.37
GLIS2	608539	100	100	98.49	91.58
GNAT1	139330	100	100	100	100
GNAT2	139340	100	100	100	100
GNB3	139130	100	100	100	99.17
GNPTG	607838	100	97.00	93.66	93.66
GPR143	300808	98.42	80.60	80.60	73.74
GPR179	614515	100	99.22	97.66	95.43
GRHL2	608576	100	100	100	97.04
GRIP1	604597	100	100	100	98.20
GRK1	180381	100	100	100	98.96
GRM6	604096	98.50	96.06	94.37	89.49
GRN	138945	100	100	100	99.41
GSN	137350	100	98.18	94.55	86.67
GUCA1A	600364	100	100	100	100
GUCA1B	602275	100	100	100	95.59
GUCY2D	600179	100	100	98.65	84.36
HARS1	142810	100	100	98.41	93.84
HCCS	300056	100	100	100	97.78
HGSNAT	610453	94.18	93.95	91.93	84.15
HK1	142600	100	100	99.45	95.93
HMGB3	300193	100	100	100	90.07

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
HMX1	142992	100	89.67	66.64	12.79
HPS1	604982	100	100	100	92.54
HPS3	606118	100	100	98.37	89.11
HPS4	606682	100	100	100	97.21
HPS5	607521	100	100	99.40	93.81
HPS6	607522	100	100	96.48	82.94
HSF4	602438	100	99.71	97.61	87.15
IDH3A	601149	100	96.43	96.43	92.37
IDH3B	604526	100	100	100	100
IFT140	614620	100	100	99.50	94.87
IFT172	607386	100	100	99.99	98.15
IFT27	615870	100	100	100	96.06
IFT43	614068	100	100	100	98.88
IFT52	617094	100	100	100	100
IFT74	608040	100	98.69	91.97	70.76
IFT81	605489	100	100	99.16	89.36
IGBP1	300139	100	100	96.65	86.72
IMPDH1	146690	100	98.20	92.23	89.64
IMPG1	602870	100	100	99.63	91.83
IMPG2	607056	100	99.67	98.29	96.20
INPP5E	613037	99.81	97.70	95.03	87.08
INVS	243305	100	99.52	98.69	96.27
IPO13	610411	100	100	99.91	96.94
IQCB1	609237	100	100	100	99.95
JAM3	606871	100	100	100	99.41
KATNIP	616650	100	99.99	98.94	94.25
KCNJ13	603208	100	100	100	100
KCNV2	607604	100	100	100	99.46
KERA	603288	100	100	100	100
KIAA0586	610178	100	100	98.11	89.90
KIAA1549	613344	96.83	96.69	96.45	93.67
KIF11	148760	100	99.50	96.77	89.01
KIF21A	608283	100	99.34	98.31	96.24
KIF3B	603754	100	100	100	99.56
KIF7	611254	99.84	96.33	91.96	81.11
KIZ	615757	100	100	99.80	94.36
KLHL7	611119	100	100	100	100
KRT12	601687	100	100	100	100
KRT3	148043	100	100	99.32	88.98
LAMA1	150320	100	100	99.99	98.15

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
LAMB2	150325	100	100	99.65	97.79
LCA5	611408	100	100	100	97.80
LCAT	606967	100	100	99.31	87.92
LEMD2	616312	100	100	100	100
LIM2	154045	100	100	100	100
LOXL3	607163	100	100	99.70	97.48
LRAT	604863	100	100	100	100
LRIT3	615004	100	100	100	100
LRMDA	614537	100	100	94.38	93.89
LRP2	600073	100	99.89	99.21	97.47
LRP5	603506	97.91	97.91	97.46	94.32
LRPAP1	104225	98.86	93.70	87.49	81.07
LSS	600909	100	99.13	98.22	90.93
LTBP2	602091	100	98.58	95.78	88.07
LYST	606897	100	99.93	99.20	94.88
LZTFL1	606568	100	97.97	88.39	76.98
MAB21L2	604357	100	100	100	99.91
MAF	177075	84.75	78.90	75.82	70.62
MAK	154235	100	99.78	95.01	93.62
MAPKAPK3	602130	100	100	100	96.73
MERTK	604705	100	100	100	98.86
MFN2	608507	100	100	100	97.40
MFRP	606227	100	100	100	97.15
MFSD8	611124	100	100	100	98.06
MIP	154050	100	100	100	99.71
MIR184	613146	No coverage data			
MIR204	610942	No coverage data			
MITF	156845	100	100	100	100
MKKS	604896	100	100	100	100
MKS1	609883	100	99.80	98.92	94.23
MLPH	606526	100	97.71	91.81	88.95
MTPP	157147	100	100	100	100
MVK	251170	100	100	100	97.30
MYO5A	160777	98.09	96.39	94.93	92.09
MYO7A	276903	100	100	98.55	95.50
MYOC	601652	100	100	100	99.49
NAA10	300013	100	99.83	90.58	57.17
NAGLU	609701	100	94.55	91.12	82.97
NBAS	608025	100	100	99.93	97.07
NDP	300658	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
NEK2	604043	100	100	100	97.12
NEK8	609799	100	100	100	96.49
NEUROD1	601724	100	100	100	100
NHS	300457	100	96.81	92.89	87.99
NMNAT1	608700	100	100	100	100
NPHP1	607100	96.88	96.88	96.88	94.79
NPHP3	608002	100	100	98.75	92.34
NPHP4	607215	100	100	99.64	93.51
NR2E3	604485	99.86	99.86	99.26	92.16
NR2F1	132890	100	97.40	92.36	82.84
NRL	162080	100	100	98.93	88.08
NYX	300278	100	100	96.49	83.04
OAT	613349	100	95.40	92.47	87.93
OCA2	611409	100	100	99.60	93.85
OCRL	300535	98.15	98.15	98.15	97.93
OFD1	300170	100	99.66	98.24	86.94
OPA1	605290	100	99.75	97.50	90.10
OPA3	606580	100	100	100	100
OPN1LW	300822	89.44	86.72	80.53	67.41
OPN1MW	300821	52.64	39.77	31.39	21.45
OPN1SW	613522	100	100	100	99.56
OPTN	602432	100	99.52	95.15	77.07
OTX2	600037	100	100	100	100
OVOL2	616441	99.83	90.88	76.08	65.97
P3H2	610341	100	100	100	89.91
P4HA2	600608	100	100	100	100
PANK2	606157	100	100	100	99.06
PAX2	167409	48.09	42.35	35.79	35.79
PAX6	607108	100	100	100	100
PCARE	613425	100	100	100	98.59
PCDH15	605514	100	100	99.60	98.53
PCYT1A	123695	100	100	100	99.64
PDE6A	180071	100	100	100	99.45
PDE6B	180072	100	100	100	98.70
PDE6C	600827	100	100	100	96.62
PDE6D	602676	100	100	100	100
PDE6G	180073	100	100	100	98.75
PDE6H	601190	100	100	89.97	50.16
PDZD7	612971	100	100	100	96.27
PEX1	602136	100	99.27	96.71	92.61

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
PEX2	170993	100	100	100	100
PEX26	608666	100	100	100	97.05
PEX6	601498	100	99.44	96.19	88.40
PEX7	601757	100	100	98.16	87.17
PGK1	311800	100	100	100	95.68
PHOX2A	602753	100	100	90.52	64.97
PHYH	602026	100	89.33	89.33	80.90
PIGL	605947	100	100	100	100
PIK3R1	171833	100	100	100	98.58
PIKFYVE	609414	100	100	100	97.78
PITPNM3	608921	98.74	96.46	92.66	87.01
PITX2	601542	100	97.48	93.41	85.17
PITX3	602669	100	100	100	87.94
PLA2G5	601192	100	100	100	100
PLK4	605031	100	100	100	97.35
PNPLA6	603197	100	99.15	96.43	90.99
POC1B	614784	100	99.46	95.22	89.78
POC5	617880	100	100	99.82	96.61
POMGNT1	606822	100	100	100	99.44
PPT1	600722	100	100	100	96.43
PRCD	610598	100	93.47	59.46	18.02
PRDM13	616741	100	96.34	91.59	83.55
PRDM5	614161	100	100	98.66	91.35
PRIMPOL	615421	100	100	99.82	91.76
PROM1	604365	97.56	97.56	96.60	92.30
PRPF3	607301	100	100	100	96.72
PRPF31	606419	100	100	100	99.86
PRPF4	607795	100	100	100	100
PRPF6	613979	100	100	100	97.47
PRPF8	607300	100	100	100	99.13
PRPH2	179605	100	100	98.00	94.26
PRSS56	613858	100	98.36	92.52	68.27
PXDN	605158	100	99.80	99.01	95.27
RAB28	612994	100	97.95	92.96	83.56
RAB3GAP2	609275	100	99.48	98.16	95.87
RARB	180220	100	100	100	97.77
RAX	601881	100	100	93.03	77.96
RAX2	610362	100	100	100	90.71
RBP3	180290	100	100	100	100
RBP4	180250	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
RCBTB1	607867	100	100	100	100
RD3	180040	100	97.76	88.32	59.52
RDH11	607849	100	100	100	95.60
RDH12	608830	100	100	99.91	94.58
RDH5	601617	100	100	100	100
REEP6	609346	100	99.93	96.22	82.47
RGR	600342	100	100	99.32	86.46
RGS9	604067	100	100	100	96.50
RGS9BP	607814	100	100	100	91.86
RHO	180380	100	100	100	100
RIMS1	606629	100	100	98.41	82.50
RIMS2	606630	100	100	97.74	77.33
RLBP1	180090	100	100	100	96.43
ROM1	180721	100	99.46	97.48	82.35
RP1	603937	100	100	100	99.66
RP1L1	608581	100	100	97.40	89.04
RP2	300757	100	100	100	92
RP9	607331	100	100	97.89	91.89
RPE65	180069	100	100	100	100
RPGR	312610	76.88	72.78	69.77	60.46
RPGRIP1	605446	100	100	99.86	98.24
RPGRIP1L	610937	99.64	97.81	94.99	91.37
RS1	300839	100	100	100	98.42
RTN4IP1	610502	100	100	100	100
SAG	181031	100	100	100	100
SALL2	602219	100	100	100	98.82
SALL4	607343	100	99.10	95.37	95.37
SAMD11	616765	80.09	71.81	65.36	55.19
SC5D	602286	100	100	100	100
SCAPER	611611	100	99.90	98.97	94.27
SCO2	604272	100	100	100	100
SDCCAG8	613524	100	100	98.97	90.75
SEMA4A	607292	100	99.86	99.22	94.42
SGSH	605270	93.52	93.52	93.52	93.52
SHH	600725	100	100	100	100
SIPA1L3	616655	100	99.38	97.47	91.42
SIX3	603714	100	100	97.64	93.34
SIX6	606326	100	100	99.81	96.40
SLC16A12	611910	100	100	100	99.31
SLC24A1	603617	100	100	98.99	91.00

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
SLC24A5	609802	100	100	100	96.13
SLC25A46	610826	100	100	95.54	92.43
SLC33A1	603690	100	100	100	98.25
SLC38A8	615585	100	100	99.14	88.80
SLC39A5	608730	100	99.78	96.76	81.32
SLC45A2	606202	100	100	100	100
SLC4A11	610206	100	100	100	98.68
SLC52A2	607882	100	100	100	100
SLC7A14	615720	100	100	100	99.18
SMOC1	608488	100	97.99	93.27	88.67
SNRNP200	601664	100	100	100	99.49
SOX2	184429	100	100	99.23	93.41
SOX5	604975	100	100	100	97.68
SPATA7	609868	100	97.79	92.05	83.41
SPP2	602637	100	100	100	100
SRD5A3	611715	100	100	98.10	93.45
SSBP1	600439	100	100	100	100
STRA6	610745	100	100	100	97.57
SUFU	607035	100	100	99.97	96.69
TACSTD2	137290	100	100	100	98.08
TAMALIN	612027	90.71	77.57	62.79	44.57
TCF4	602272	100	99.49	98.62	94.81
TCOF1	606847	100	99.42	96.89	87.48
TCTN1	609863	100	100	100	100
TCTN2	613846	100	100	100	96.98
TCTN3	613847	100	100	100	98.86
TDRD7	611258	99.94	98.98	98.41	96.18
TEAD1	189967	100	100	100	99.80
TEK	600221	100	100	100	99.97
TENM3	610083	100	100	99.88	98.54
TFAP2A	107580	100	100	100	94.92
TGFBI	601692	100	99.12	96.30	93.14
TGIF1	602630	100	100	100	100
TIMM8A	300356	100	100	100	90.18
TIMP3	188826	100	100	98.50	85.95
TMCO3	617134	100	99.27	97.45	90.22
TMEM107	616183	100	100	100	100
TMEM126A	612988	100	100	100	100
TMEM138	614459	100	100	100	100
TMEM216	613277	100	100	100	100

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
TMEM231	614949	100	100	100	97.47
TMEM237	614423	100	100	96.66	94.14
TMEM67	609884	100	100	99.45	90.94
TMEM98	615949	100	100	100	100
TOGARAM1	617618	100	100	99.18	95.78
TOPORS	609507	100	100	100	98.90
TPP1	607998	100	100	98.00	93.71
TRAF3IP1	607380	100	99.05	94.07	74.99
TREX1	606609	100	100	100	100
TRIM32	602290	100	100	100	100
TRNT1	612907	100	95.77	89.81	87.03
TRPM1	603576	100	99.64	99.01	97.31
TSPAN12	613138	100	100	100	97.82
TTC8	608132	100	100	100	100
TLL5	612268	100	99.93	99.11	95.63
TUB	601197	100	99.85	97.17	89.81
TUBA3D	617878	100	100	100	100
TUBB3	602661	100	99.30	94.62	94.62
TUBB4B	602660	100	100	100	98.48
TUBGCP4	609610	100	100	99.79	97.71
TUBGCP6	610053	100	100	100	98.75
TULP1	602280	100	96.57	83.49	59.09
TYR	606933	100	100	100	99.76
TYRP1	115501	100	100	100	100
UBIAD1	611632	100	100	100	100
UNC119	604011	100	99.82	97.20	89.21
UNC45B	611220	100	100	100	98.64
USH1C	605242	100	100	100	95.40
USH1G	607696	100	100	100	100
USH2A	608400	100	100	100	99.46
USP45	618439	100	98.81	95.17	86.84
VAX1	604294	100	98.82	92.55	61.44
VCAN	118661	100	100	100	100
VIM	193060	100	100	100	99.87
VSX1	605020	100	93.65	81.85	69.92
VSX2	142993	100	100	99.79	89.26
WASHC5	610657	100	100	100	98.17
WDPCP	613580	100	99.77	97.50	94.38
WDR19	608151	100	100	99.54	96.20
WDR36	609669	100	100	100	98.38

HGNC approved gene symbol	OMIM gene ID (active link to omim.org)	% covered ≥10x	% covered ≥20x	% covered ≥30x	% covered ≥50x
WFS1	606201	100	100	99.45	97.03
WHRN	607928	100	100	99.97	95.53
WRN	604611	100	99.89	97.79	92.36
YAP1	606608	100	98.30	95.04	88.85
YME1L1	607472	100	99.23	96.25	84.87
ZEB1	189909	100	99.79	97.88	90.42
ZNF408	616454	100	100	100	99.27
ZNF423	604557	98.51	98.51	98.51	98.06
ZNF469	612078	100	99.63	98.38	93.54
ZNF513	613598	100	100	98.15	92.28
ZNF644	614159	100	100	100	99.85

- OMIM release used: 23-9-2021

- The statistics above are based on a set of 150 samples

- % Covered 10x , 20x, 30x and 50x describes the percentage of a gene's coding sequence (±10bp flanking introns) that is covered at least 10x, 20x, 30x or 50x